


Just set the cup in place. Comfortable urination in your usual posture.

Urine Raku-Ryu Holder




Instructions For Use

Step1. Preparation

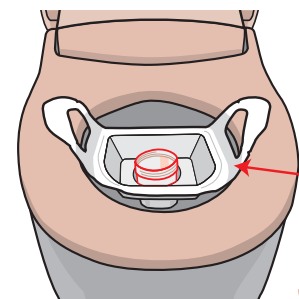

1. Adjust the position and use the two adhesive tapes on the back to fix the cup at a lower position than the toilet seat surface.


2. Do not flush the tape liners. Please throw them in the trash.

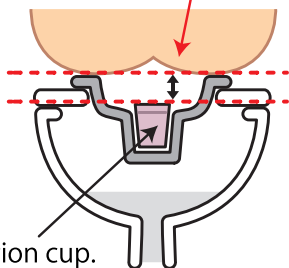


3. Set the urine collection cup.

Step2. Retrieve


1. After urination, drop the toilet paper into the toilet bowl.
2. Remove the urine collection cup, close the lid, and wipe off any urine with toilet paper.

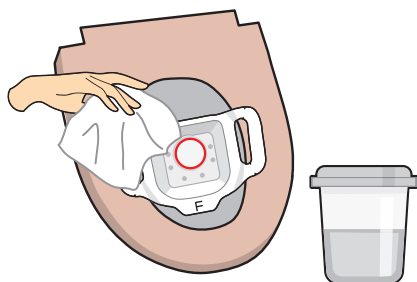

Step3. Flushing

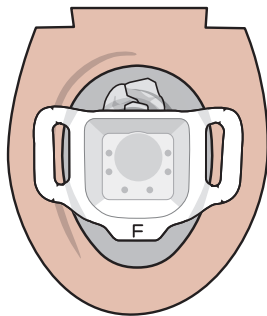

1. Flush the toilet paper as is.

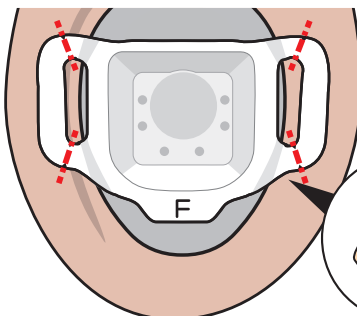

2. Detach along the perforation and drop into the toilet. Fold the adhesive tape parts inward and drop them in the toilet as well.



3. Once the tank is refilled, flush the remaining product.

Dispose of all items other than the retrieval cup according to your local regulations.

Warnings

- This product is not edible. Do not ingest.
- This product is designed exclusively for urine sample collection. Do not use for any other purpose.
- This product is not hypoallergenic. Individuals with allergies should evaluate its use at their discretion.
- Improper uses of this product include uses that do not conform to the following usages.

Usage Notes

- Do not flush this product together with accumulated toilet paper in the toilet bowl. It may cause clogging.
- Ensure the toilet tank is filled with sufficient water before flushing.
- Damage or malfunction of the toilet or surrounding equipment caused by improper use of this product is not covered under warranty.
- If the product or toilet seat is wet, the product may tear during use.
- Use only one sheet at a time. Using multiple sheets simultaneously may clog the drain.
- Do not shake the product violently with urine inside
- Do not leave the product in the toilet for an extended period. Flush promptly after use.
- Keep out of reach from small children.
- If a coating is applied to the toilet seat, the adhesive part may peel off easily.
- Do not flush the protective seat, as it may cause clogging.
- Do not use this product in toilets with drain pipes with diameters smaller than 75 mm (3 in). If used in such toilets, clogging may occur, potentially causing contamination of the toilet bowl or floor. Such damages are not covered under warranty.
- Do not use bidet or cleaning functions while this product is in use.

Storage Instructions

- Do not store in high temperature or high humidity environments. Avoid using or storing near fire.
- For long-term storage, avoid direct sunlight, high temperatures, and high humidity.

Warranty Period

- The product warranty period is one year from the date of purchase.
- If the product is defective, it will be replaced free of charge within the warranty period.

Warranty Coverage

- The above warranty applies only to the product itself. The manufacturer or seller assumes no responsibility for damages caused by the use of this product, including damage to other products or property.

